

Glossary

ARMREST TYPES

Full Length

Provide more support for easier ingress and egress.

Desk Length

Allow for easy fit under most desks and tables.

RIGGING STYLES

Swing-Away Footrests

Provide foot support while seated and rotate out of the way for easier transfers.

Elevating Legrests

Offer leg support with comfortable calf pads and allow angle-of-elevation adjustment.

CHASSIS FINISH

Chrome

Resistant to corrosion, easy to clean, and sturdy.

Powder-Coated Silver Vein Finish

Smooth finish with an extremely durable coating.

CHASSIS TYPE

Standard

Reclining

Measuring Guidelines

A wheelchair user's health and wellness is dependent upon achieving the proper fit in a new wheelchair. An improper fit can cause posture issues, breathing problems, pressure ulcers, and discomfort. Make sure the user is seated in an upright position before taking any measurements. Once you have taken the resident's measurements, use the dimensions to determine the wheelchair that will best fit the resident's needs.

Seat Depth Measurement

Measure from the back of the hip to the back of the knee. Subtract one inch from this measurement to determine the best wheelchair seat depth dimension for your resident.

Seat Width Measurement

Measure between the widest point on the hips or thighs. Use this measurement to determine the appropriate wheelchair seat width for your resident.

Footrest/Legrest Length Measurement

Measure from the back of the knee to the heel of the foot. Elevating legrests are always suggested when residents are required to raise their legs regularly.

Seat to Floor Height Measurement

Measure the distance between the back of the knee and the heel of the foot. This measurement will determine if a dual-axle wheelchair with varying seat-to-floor heights would be appropriate for the resident. If the resident will be self-propelling, a hemi seat height is recommended.

Back Height Measurement

Measure from the resident's collarbone to the seat. Use this measurement to determine the appropriate wheelchair back height. Taller back heights may be required for residents who need upper trunk support.



Relia+ Products Provide Quality Care Solutions

Performance, reliability, and value are the cornerstone qualities of our growing family of exclusive Relia+ products. With sought-after product features, we continue to build our offering along with your trust. Our selection, made to exceed professional expectations and increase resident satisfaction, now includes mattresses, wheelchairs, wheelchair cushions, and exam gloves.

See our full-line Catalog V65, pages 1674-1678, and our Healthcare Catalog V10, pages 10-14, for our complete selection of wheelchairs.



TYPE	BRAND	PRODUCT NAME	WEIGHT CAPACITY	SEAT WIDTH	SEAT DEPTH	BACK HEIGHT	ARMREST STYLE	RIGGING STYLE	CHASSIS FINISH	ADJUSTABLE HEIGHT	WARRANTY	FREE DELIVERY	PART #
STANDARD	Relia+	Standard Wheelchair	300 Lb	18"	16"	18"	Desk Length	Elevating Legrests	Powder-Coated Silver Vein	Yes	3-Year Limited	Yes, Next Day	680615: See V10, Page 10
		Standard Wheelchair	300 Lb	18"	16"	18"	Full Length	Elevating Legrests	Powder-Coated Silver Vein	Yes	3-Year Limited	Yes, Next Day	680616: See V10, Page 10
		Standard Wheelchair	350 Lb	20"	16"	18"	Desk Length	Elevating Legrests	Powder-Coated Silver Vein	Yes	3-Year Limited	Yes, Next Day	680617: See V10, Page 10
		Standard Wheelchair	350 Lb	20"	16"	18"	Full Length	Elevating Legrests	Powder-Coated Silver Vein	Yes	3-Year Limited	Yes, Next Day	680618: See V10, Page 10
	INVACARE Yes, you can.®	Tracer EX2†	250 Lb	18 Or 20"	16"	18"	Full Length Or Desk Length	Legrest Or Footrest	Powder-Coated Silver Vein	Yes	5-Year On Frame And Cross Braces	No	See V10, Page 12
		Tracer IV†	350 Lb	20 Or 22"	18"	18"	Full Length Or Desk Length	Legrest Or Footrest	Powder-Coated Silver Vein	Yes	5-Year On Frame And Cross Braces	No	See V10, Page 13
RECLINING	Relia+	Reclining Wheelchair	300 To 350 Lb	18 Or 20"	16"	33"	Desk Length	Elevating Legrests	Powder-Coated Silver Vein	Yes	3-Year	Yes	554001, 554002: See V10, Page 11
	INVACARE Yes, you can.®	Tracer SX5†	250 Lb	18 Or 20"	16"	33"	Desk Length	Legrest Or Footrest	Powder-Coated Silver Vein	Yes	5-Year On Frame And Cross Braces	No	See V10, Page 13
BARIATRIC	Relia+	Bariatric Wheelchair*	450 Lb	22 Or 24"	18"	18"	Full Length	Elevating Legrests	Powder-Coated Silver Vein	Yes	Limited Lifetime	Yes	680619, 680620: See V10, Page 14 Or 60
	GENDRON Bariatric Rehab Leadership	Regency XL Bariatric Wheelchair†	600 Lb	24, 36, Or 28"	18"	18"	Full Length	Swing-Away Footrests	Chrome	Yes	Limited Lifetime	No	676420, 676421, 676422: See V10, Page 14 Or 60
		Regency XL 2000†	750 Lb	28, 30, Or 32"	20"	19-1/2"	Full Length	Swing-Away Footrests	Powder-Coated Black	Yes	Limited Lifetime On Frame And Cross Braces	No	676423, 676424, 676425: See V10, Page 14 Or 60
BARIATRIC RECLINING	GENDRON Bariatric Rehab Leadership	Regency XL 2000 Recliner Wheelchair†	750 Lb	28, 30, Or 32"	20"	36"	Desk Length	Elevating Legrests	Powder-Coated Black	Yes	Limited Lifetime On Frame And Cross Braces	No	676506, 676507, 676508: See V10, Page 14 Or 60

* 1-2 Days Lead Time † 5-8 Days Lead Time ‡ 5-7 Days Lead Time